

Esatto Laser



ReviLift Hifu


Face lifting & Body slimming

HIFU Handle




Face

1.5mm 2.0mm 4.5mm 3.0mm 4.5mm


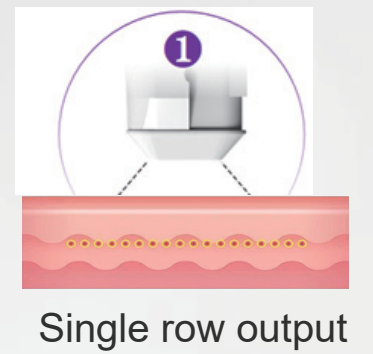


Body

6.0mm 9.0mm 12.0mm

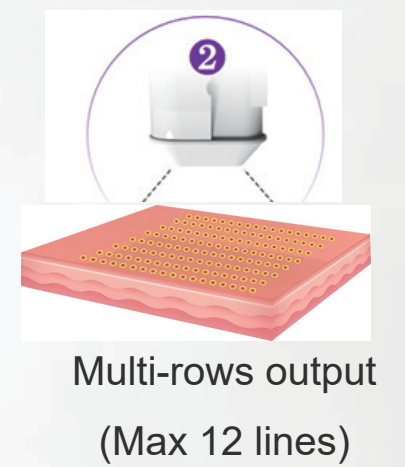

▶ Different size cartridges for your choice

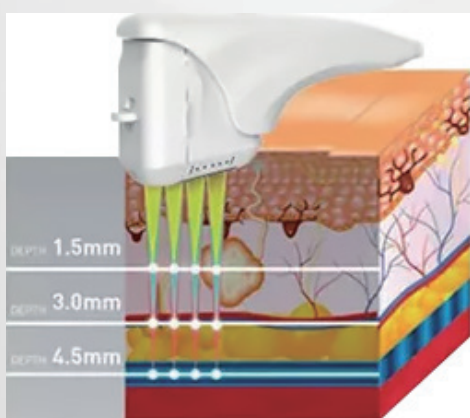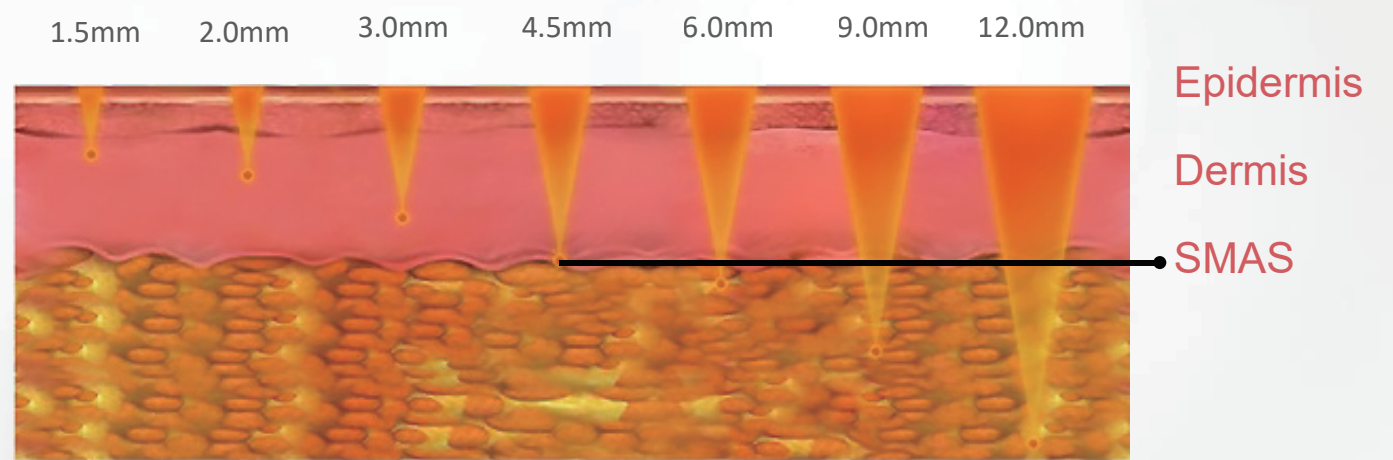


▶ HIFU – High-Intensity Focused Ultrasound Principle

HIFU works by using ultrasound energy to penetrate deep into the skin surface layer, reaching the superficial muscular aponeurotic system (SMAS layer) to stimulate collagen production and tighten the skin.

Hi 360° Handle

Work in the small corners of nose mouth and eyes

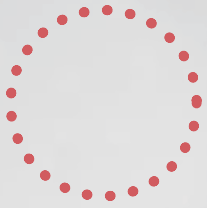

360°



180°



Point



1.5mm



3.0mm

**40,000
shots**



Small & Sensitive Areas

- Eyes
- Nose
- Forehead
- Crow's feet
- Nasolabial Folds
- Around mouth



Advantages

Combine Single-line + Multi-line together

Our hifu handle with both Single line cartridges can use for narrow areas and stay treatment, the multi-line cartridges can used for big areas and sliding treatment

Painless with Hi-360 handle

The 360handle can used for small areas, like eyes around ,nose around, and this small handle totally painless.


Immediate effect

Obvious results can be obtained after one treatment

Convinent And Comfortable

Combine 2 handles can provide full face solution, meet different customers' needs


Function

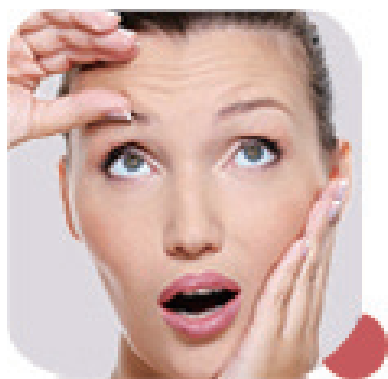

Face lifting



Anti-wrinkle

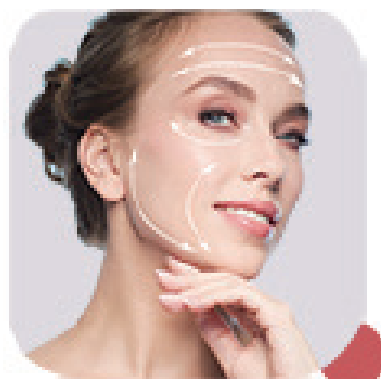

Skin tightening



Body shaping








Face



Body

Treatment parameters


Working handle	Cartridge/mode	Areas	Parameters	How many shots does each treatment area need
HIFU handle	Single-line cartridge	Cheek	0.3J to test Parameter range:0.7-1.2j 3.0mm	Face and neck : 300-400shots Abdomen: 40 shots Arm:200-300 shots Leg:300-400 shots
		Eye	0.3J to test Parameter range:0.7-1.2j 2.0mm	
		Forehead	0.3J to test Parameter range:0.7-1.2j 1.5mm	
		Then use the multi-line cartridge Pitch:1.5row/6 0.8-1.5j slidingway From the first line to do the sliding way		
	Multi-line cartridge	Neck	0.5J to test Parameter range:1.0-1.5j row:6 4.5mm Stay way in H orizon way, then use sliding way	
		Arm	1.1-1.5J 6mm 8-10lines from bottom to top,stay in horizon way, then sliding way	
		Calf	6.0mm 1.5-2J 9-12 lines	
Thigh		6.0/9.0mm 1.5-2J 9-12 lines		
HI 360° handle	Mode1	Eye round	0.5J to test if the customer can accept parameter range:0.81.5j---Stay way, then sliding way, can get a better treatment result	100-150 shots
		Nose round		
	Mode2	Eye coner		
		Mouth comer		
Mode3	Upperlip	0.5-0.7J; sliding way		

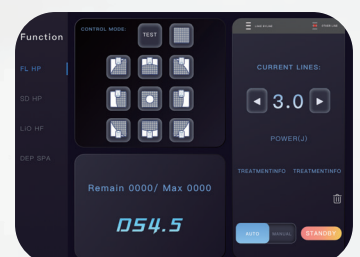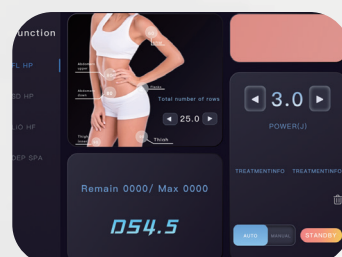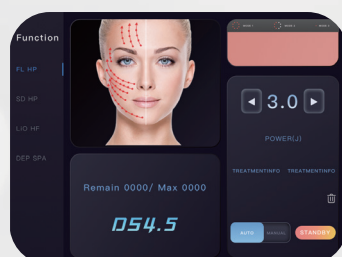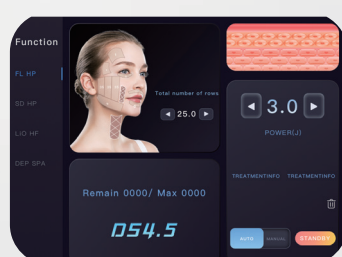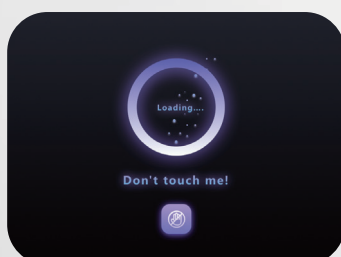

Specification

Type	High Intensity Focused Ultrasound
Screen	15 inch color touching screen
Energy	0.1-3.0J
Operate mode	Auto and manual
Motor	Japan motor
Working Handles	12D + HI 360° handle
Cartridges	10 Pcs
Cartridge frequency	4-7 Mhz
Voltage	AC 110V/220V 50-60Hz



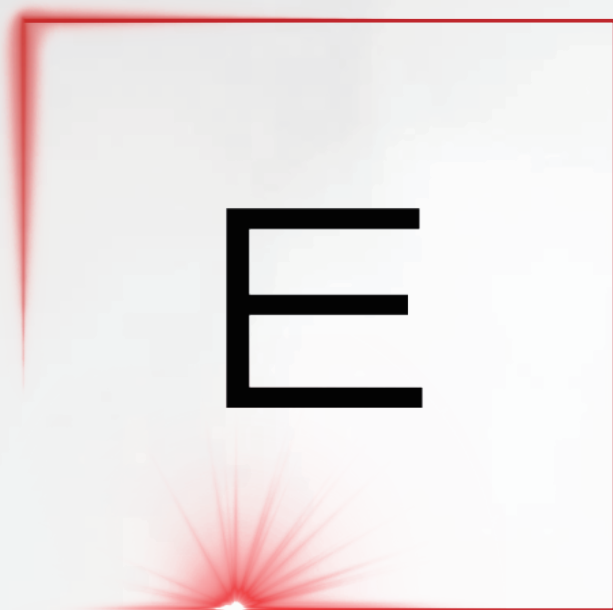

ESATTO LASERSTM
Aesthetic Solutions